DISPOSABLE PROTECTIVE CLOTHING
TYPE 3-B, TYPE 4-B, TYPE 5-B, TYPE 6-B
REF: CA-403

OWNER NU: 10075476

LOT : 89E3GH007
MANF. DATE : 12/2020
EXP. DATE : 12/2023

Product Description:

Type 3,4,5,6 CE Certified Protection Coveralls, blue tape heat sealed seams

Coveralls for Protection against spills and viruses
isolation room coveralls

*Fabric Type: Non-woven fabric with PE film breathable

*Style : Coverall style with elastic with tumb loop

*Style : Coverall style with elastic cuff, zipper clousre, elastic hood

* Feature: Soft, light, non-toxic, durable, eco-friendly

* Usage: Prevent and isolate liquid,dust, virus invading

*Application: Hospital, chemical industry, food, industry, etc

PROTECTION LEVELS

Type 3

Type 4

Type 5


Type 6


EN 14126

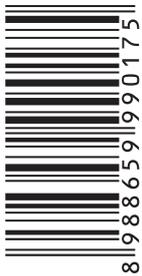EN 1149-5: 2018

CATEGORY III

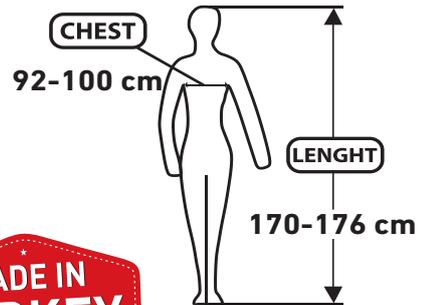Manf: STERIL SAGLIK URUNLERI
ZIYAPASA MH 67132 SOK GULMEN APT
ASMA K. D. 1
SEYHAN/ADANA- TURKEY
01001

www.sterilsaglik.com
info@sterilsaglik.com

Main Use Areas

MICROSAFE CA-403 Protective Coveralls Type 3-B,4-B, 5-B, 6-B protect workers from hazardous material particles (Type 5-B) and splashes of chemical fluids (Type 6-B) and biological hazardous (Type 3-B, Type 4-B)

Restrictions Of Use

Care should be taken when removing the dirty or liquid-exposed coveralls and any contact with hazardous materials must be avoided. It is designed for extremely hazardous conditions and toxic chemicals.

If the coveralls are used with other personal protectors, the sleeve cuffs must be tied to the gloves, the ankles to the boots and the respirator (mask) to the head beforehand.

The suitability of this combination is at the discretion of the user. The static electricity on the body must be grounded properly by the person who is to wear the coverall. When grounding, the resistance on the person must be less than 108 ohms. These conditions can be easily created by wearing suitable shoes. The usage time of the product and the effect of the working environment on the comfort of use should be planned in advance.

How to wear

Make sure that your environment is clean before putting on the coverall. Open the packaging properly without damaging the product. Open the zipper of the coverall and put it on, feet first. Do not crumple the coverall too much while doing this. After putting your body and arms into the coverall, put the hood on your head and close the zipper. Remove the paper of the adhesive tape at the zipper area and stick it to the coverall, taking care not to fold it. In this way, the permeability of the zipper will be prevented.

Storage / Final Use

It is recommended to store the box between 15 - 25 ° C and to protect it from sunlight. When stored under suitable conditions, it is recommended to use it within 3 years from the date of production.

Disposal and recycling

Non-contaminated products can be treated as general garbage or recycled. Contaminated products, on the other hand, must be treated as harmful wastes and disposed in accordance with the rules specified by law.

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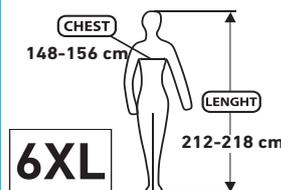
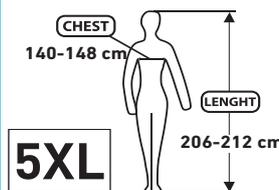
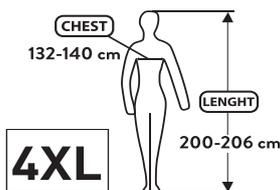
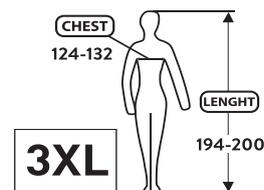
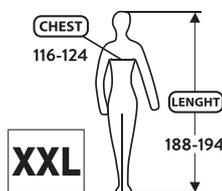
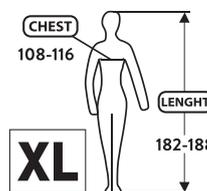
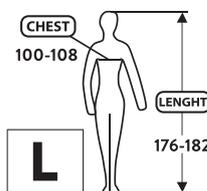
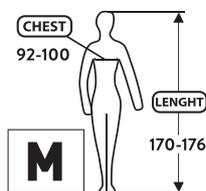
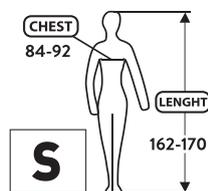
SAFETY INSTRUCTION

Check the coverall for manufacturing defects such as cracks, holes and running stitches before use. Do not use faulty and/ or contaminated coveralls.

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ATTENTION! Playing with the bag is dangerous and can lead to suffocation. Keep the bag away from children.

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